

Instruments Cleaner

The Instruments Cleaner cleans and sterilizes all surgical instruments, masks, and launders the surgical towels.

Instruments Cleaner Instructions

1. **Clean and sterilize** the surgical instruments:
 - Scrub used instruments in Alconox and hot water, paying special attention to the teeth and joints where blood collects.
 - Place instruments into ultrasonic cleaner for 10 minutes.
 - Rinse in water container in front of ultrasonic cleaner.
 - Lay instruments on towel in single layer and spray with instrument milk.
 - Sort the instruments in the trays by type.
2. **Wash, dry and fold** surgical linens:
 - Flatten half the towels for wrapping spay and neuter packs.
3. **Place re-wrapped packs in the autoclave.** Set the machine. Wait until items have cooled before storing.

All Station Images



Station Closing and Cleanup

When the clinic is concluded:

- Inventory the remaining supplies.
- Clean the area and dispose of all trash.
- Wash all tables